## IMPORTANT!

WHEN CHARGING, DO NOT TURN ON THE SWITCH BEFORE THE CHARGER IS CONNECTED TO THE CHAIR.

THE BATTERY SHOULD BE CHARGED 24 HOURS THE FIRST TIME.

WHEN THE BATTERY HAS RUN DOWN THERE IS A BEEP SOUND FROM THE CONTROL BOX WHEN THE ELECTRICAL PARTS ARE IN USE.

WHEN THE BEEP SOUND COMES, YOU HAVE TO RECHARGE THE BATTERY AS IT CAN BE DAMAGED IF YOU LET IT RUN 100 % OUT OF POWER.

(IF THE BATTERY HAS RUN OUT OF POWER, YOU CANNOT USE THE ENGINE BY PUTTING THE CHARGER IN THE WALL PLUG AND THE CHAIR. A FULL RECHARGE TAKES TYPICALLY 8 HOURS.)

IF THE LIGHTBOLT ON THE CHARGER LIGHTS
GREEN WHEN CONNECTED, IT IS DUE TO A
MALFUNCTION OF THE BATTERY, AND THEREFORE
IT NEEDS TO BE CHANGED.

DO NOT CHARGE THE CHAIR WHILE BEING USED IN AREAS WITH WATER.

DO NOT USE THE ELECTRICALL FUNCTIONS ON THE CHAIR WHILE SPRAYING WITH WATER.

#### M2-200 KG'S MOTOR

2/4

USERS GUIDE: SHOWER/COMMODE CHAIR M2. CHAIR FOR WASHING/TOILETING PURPOSES

#### **LOCKING THE WHEELS:**



BEFORE YOU USE THE CHAIR FOR BATHING/SHOWERING YOU CAN APPLY THE BRAKE ON THE FRONT WHEELS. THE DRIVING WHEELS ARE AUTOMATICLY LOCKED WHEN YOU ARE NOT DRIVING.

#### MOUNTING OF FOOT RESTS ON THE SHOWER/COMMODE CHAIR:




ADJUST THE HEIGHT OF THE FOOTRESTS BY TAKING THE CLIPS OUT, PLACING THE FOOTRESTS IN THE WISHED POSITION AND AT LAST PUT THE CLIPS BACK AGAIN.

#### TO PUT SEAT ON:



MAKE SURE THAT THE BACK EDGE OF THE SEAT IS FITTED AROUND THE FLAT METAL BAR AT THE BACK.

#### **CLEANING:**

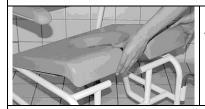


TO MAKE THE CLEANING TASK EASIER YOU CAN CLICK THE SEAT ON AND OFF.

#### **REPLACEMENT OF FRONT WHEEL:**



TAKE OUT THE CLIPS AND THE WHEEL AND PUT THE NEW WHEEL ON.

## 1) CHARGING THE BATTERY

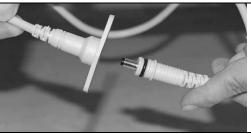

WITH THE ONE HAND, YOU GRAB AROUND THE CHARGING PLUG (SOCKET) ON THE CHAIR AND WITH THE OTHER HAND YOU PULL OUT THE CONNECTING CABLE FOR THE BATTERY.

## 2) CHARGING THE BATTERY



WITH THE ONE HAND, YOU GRAB AROUND THE CHARGING PLUG (SOCKET) ON THE CHAIR AND WITH THE OTHER HAND, YOU PUSH THE PLUG FROM THE TRANSFORMER IN PLACE.

#### 3) CHARGING THE BATTERY



DEPENDING ON REGION PLUG TYPE 1, 2 OR 3 SLIDES INTO THE CHARGER.

AFTERWARDS YOU PUT THE TRANSFORMER IN THE SOCKET AND TURN ON THE POWER.  $\,$ 

A FULL CHARGE TAKES APPROXIMATELY 8 HOURS. WHEN THE BATTERI IS RUNNING OUT OF POWER YOU WILL HEAR A "BEEP" SOUND.

WHEN THE BATTERY IS BEING CHARGED, THE LIGHT BOLT ON THE CHARGER LIGHTS ORANGE.

WHEN THE BATTERY HAS BEEN FULLY CHARGED, THE LIGHT BOLT ON THE BATTERY WILL LIGHT GREEN.

NOTE: IF THE LIGHT BOLT ON THE CHARGER LIGHTS GREEN, WHEN IT IS BEING CONNECTED, IT IS DUE TO A

#### M2-200 KG'S MOTOR

#### 3/4

### **USING THE CHAIR: USE OF JOYSTICK:**



THE CHAIR IS EASILY CONTROLLED BY PUSHING THE JOYSTICK IN THE DIRECTION YOU WANT TO DRIVE. WHEN YOU LET THE JOYSTICK GO THE CHAIR STOPS (IT WORKS LIKE A BRAKE).

#### PLACING THE JOYSTICK:



THE JOYSTICK CAN BE PLACED AT THE FRONT SO THE USER CAN CONTROL IT OR AT THE BACK SO THE HELPER CAN CONTROL IT. YOU FASTEN IT TO THE CHAIR BY TIGHTENING THE FINGER SCREW.



#### **CHARGING THE BATTERIES:**



THE 6 LIGHTS INDICATES HOW MUCH POWER THERE IS LEFT IN THE BATTERIES. WHEN THE GREEN LIGHTS STOP LIGHTING YOU HAVE TO RECHARGE THE CHAIR. PLEASE NOTICE:

IF YOU USE UP ALL THE BATTERI POWER WITHOUT RECHARGING THE BATTERIES WILL BE DAMAGED AND WILL HAVE TO BE REPLACED.

#### **DRIVING SPEED:**



YOU CONTROL THE DRIVING SPEED OF THE CHAIR BY TURNING THE NOB TO THE LEFT FOR FASTER AND TO THE RIGHT FOR SLOWER.

#### HORN:



THE CHAIR IS EQUIPPED WITH A HORN, WHICH IS ACTIVATED BY PRESSING THE HORN BUTTON IN FRONT OF THE JOYSTICK.

## **LOCKING MECHANISM:**



IT IS EASILY LOCKED (WHEN THE CHAIR IS ON) BY PRESSING AND HOLDING THE ON/OFF BUTTON FOR A FEW SECONDS.

aFTER A FEW SECONDS THE LIGHTS IN THE DISPLAY FLAS AND THE HORN IS BEEPING. IT IS NOW LOCKED.

TO UNLOCK IT YOU PRESS THE ON/OFF BUTTON AND PRESS THE HORN BUTTON 2 TIMES WITHIN 10 SECONDS.

#### M2-200 KG'S MOTOR

4/4

#### **ARMRESTS:**



TO DISASSEMBLE THE ARMREST YOU HAVE TO FIT THE HEAD OF THE BOLT TO WHERE THE ARMREST IS MILLED OUT AND PULL IT OFF.

#### IMPORTANT:

IF THE PROTECTION BOXES AROUND THE ELECTRICALL PARTS ARE IN ANY WAY DAMAGED THEY HAVE TO BE REPAIRED WITH ORIGINAL PARTS BEFORE THE CHAIR CAN BE USED AGAIN. OTHERWISE WATER CAN GET INTO THE ELECTRICAL PARTS AND DAMAGE THESE.



WHEN SLIDING THE BUCKET OF THE CHAIR, IT IS IMPORTANT TO SLIDE IT STRAIGHT BACKWARDS, ALONG THE SEAT. OTHERWISE IT CAN BE VERY DIFFICULT TO GET THE BUCKED OF THE CHAIR.


#### WARNING



DO NOT SIT ON THE OUTER 10 CM OF THE SEAT THAT THE CHAIR CAN TIP FORWARD.

# DO NOT STAND ON THE FOODRESTS



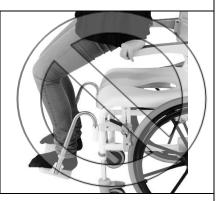

Article. No. 310274/310275

## Maximum user load 200 kg

2 year's complaint right against manufacturing errors.

1 year on batteries

## Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2-Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that normal wear is not included in the warranty.

Cleaning: The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

Spare parts M2-200 KG-MOTOR			
Article. No.:	Description	Article. No.:	Description
800194	Joystick console	310344	Pu back, grey
800195	Back wheel	310063	Screw for back, 4 pcs.
800196	Protection box, right hand side back wheel	800155	125 mm directional controlling wheel with massive leg and 1 clips
800197	Protection box, left hand side back wheel	310073	Plastic ferrule for flat armrest
800198	Control box	310074	Plastic ferrule for round armrest
800130	Charger	800245	Grey ferrule for flat oval armrest
800129	Batteri	800244	Plastic ferrule for handle
310354	Intimate seat for M2 200 kg's, grey	310076	Nylon bush for armrests, 1 pcs.
310343	Cover for intimate seat, grey	800082	Bolt for armrests, 1 pcs.
800073	Armrest for M2 200 kg., Set with 2 pieces	310065	Clips for height adjustment, grey
800181	P.U. foam cushion for armrest. One unit. Grey	310165	Clips for height adjustment, magenta
800012	Screws for P.U. foam cushion for armrest. 2 pcs.		

CE

This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.

DJMed Pty Ltd

Email info@djmed.com.au Web djmed.com.au

Mail PO Box Q541, Queen Victoria Building NSW 1230

09.10.15