IMPORTANT!

WHEN CHARGING, DO NOT TURN ON THE SWITCH BEFORE THE CHARGER IS CONNECTED TO THE CHAIR.

THE BATTERY SHOULD BE CHARGED 24 HOURS THE FIRST TIME.

WHEN THE BATTERY HAS RUN DOWN THERE IS A BEEP SOUND FROM THE CONTROL BOX WHEN THE ELECTRICAL PARTS ARE IN USE.

WHEN THE BEEP SOUND COMES, YOU HAVE TO RECHARGE THE BATTERY AS IT CAN BE DAMAGED IF YOU LET IT RUN 100 % OUT OF POWER.

(IF THE BATTERY HAS RUN OUT OF POWER, YOU CANNOT USE THE ENGINE BY PUTTING THE CHARGER IN THE WALL PLUG AND THE CHAIR. A FULL RECHARGE TAKES TYPICALLY 8 HOURS.)

IF THE LIGHTBOLT ON THE CHARGER LIGHTS
GREEN WHEN CONNECTED, IT IS DUE TO A
MALFUNCTION OF THE BATTERY, AND THEREFORE
IT NEEDS TO BE CHANGED.

DO NOT CHARGE THE CHAIR WHILE BEING USED IN AREAS WITH WATER.


DO NOT USE THE ELECTRICALL FUNCTIONS ON THE CHAIR WHILE SPRAYING WITH WATER.

User guide for Shower/Commode chair M2 EI-Tip with Joystick. Intented for use a bathing, showering and toilet visit.

(2/7)

Mounting the back:

Attach the back into the frame.

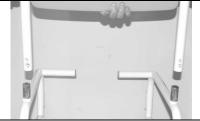

Assembly the back:

Insert the 4 screws and tighten with an Allen wrench. Make sure that <u>all</u> 4 screws are securely tightened.

Remember to retighten the screws after a month. The screw shall be discarded after the mounting and you shall use the screws to the next mounting.

(Art. No 800290)



Mounting/Disassembly of armrest

Pull the snap release and then you can tilt the armrest.



Directional controlled wheel



Before you use the chair for bathing/ showering, you must make sure you apply the brake on all four wheels.




The right front wheel can be locked the direction. Use of this makes it easier to start and moving the chair.

Continues on page 3

User guide for Shower/Commode chair M2 EI-Tip with Joystick. Intented for use a bathing, showering and toilet visit.

Back adjustment

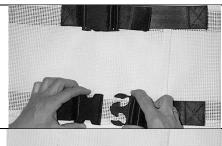




You open the back by pressing the two plastic taps with your left hand, and then with your right hand you pull the buckle out.

The depth of the back is adjusted by pulling the webbing and then tighten it.

Care: Washable at maximum 80 degrees.

Mounting the footrests

The height of the footrests are adjusted by taking the clips out and subsequently placing the footrests in a suitable height and then reinserting the clip. Placing the footrests on one side: You lift the footrests slightly and swing them to either side you want. If you want the footrests to occupy lesser space you are able to tilt them.

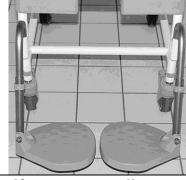

Calf support adjustment

The calf support is fastened with velcro tape on the backside.

The depth and the Calf support is adjusted by fastening the velcro tape further in or further out.

Car: Washable at maximum 80 degrees.



The pan/bucket


When sliding the bucket of the chair, it is important to slide it straight backwards, along the seat. Otherwise it can be very difficult to get the bucket of the chair.



Side support adjustment

The side supporters are clicked into place in the square pipes on the side of the chair.


The depth is adjusted by loosening the finger screw and then tightening it again once the height is right. You can also fasten the side supporters so that the cushions are turned horizontally if there is a need for that.

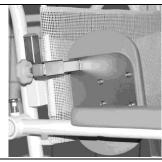

Continued on page 4

User guide for Shower/Commode chair M2 El-Tip with Joystick. Intented for use a bathing, showering and toilet visit. (4/7)


Adjusting the headrest (Art. No. 310221)

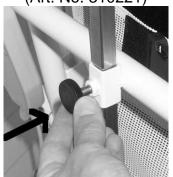

The height of the headrest is adjusted by loosening the finger screw and tightening it again when the height is right.

Use of chair in negative

Use of chair in negative position can cause that the user will fall forwards.

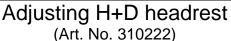



The height and depth of the headrest is adjusted by loosening the two finger screws and then tightening them again once they are in the right position.



Tilting

For going up or down the chair is tilted by using the remote-control.

If the user is not able to hold on to the armrest by himself, the helper must lay the users hands on his lap before tilting the chair.



Charging the battery

Depending on region plug type 1, 2 or 3 slides into the charger.

Afterwards you put the transformer in the socket and turn on the power.

A full charge takes approximately 8 Hours.

When the batteri is running out of power you will hear a "beep" sound.

When the battery is being charged, the light bolt on the charger lights orange.

When the battery has been fully charged, the light bolt on the battery will light green.

Note: If the light bolt on the charger lights green, when it is being connected, it is due to a malfunction, and it needs to be changed.




Continued on page 5


User guide for Shower/Commode chair M2 EI-Tip with Joystick. Intented for use a bathing, showering and toilet visit.


(5/7)

Charging the battery

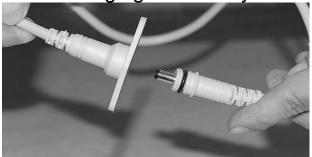


With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand you pull out the connecting cable for the battery.

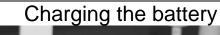

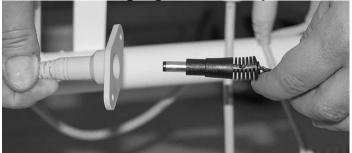

With the one hand, you grab around the charging plug (Socket) on the chair and with the other hand, you push the plug from the transformer in place.

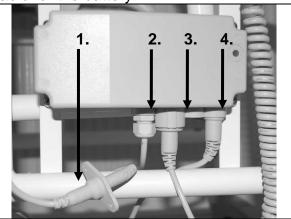

Use of joystick

The chair is easily controlled by pushing the joystick in the direction you want to drive.

Cables

- 1. Plug for battery
- 2. Cable for battery
- 3. Cable for remote control
- 4. Cable for actuator ("engine")



Placing the joystick



The joystick can be placed at the front so the user can control it or at the back so the helper can control is. You fasten it to the chair by tightening the finger screw.

Continued on page 6

User guide for Shower/Commode chair M2 EI-Tip with Joystick. Intented for use a bathing, showering and toilet visit. (6/7)



Driving speed

You control the driving of the chair by turning the turtle to slow down or turning the rabbit to go faster.



The 6 lights indicates how much power there is left in the batteries. When the green light stops you have to recharge the chair.

Please notice! If you use up all the battery power without recharging the batteries will be damaged and will have to be replaced.



The horn

The chair is equipped with a horn which is activated by pressing the horn button in the front of the joystick.



The brake

When you let the joystick go, the chair will stop (It works like a brake)



WARNING




Do not sit on the outer 10 cm of the seat that the chair can tip forward

 \leftarrow

Do not stand on the footrests.



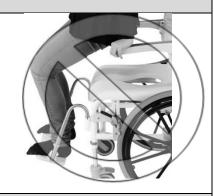

Art. No.:

Standard seat: 310204/310204-B

Max. Load 140 kg

Attention! Push the Clips to the bottom when adjusting heights.

2 Years complaint rights against manufacturing errors.

Terms of claim for frame.

HMN grants 5 years claim rights against frame break-down for all M2 Shower/commode chairs and stainless steel shower stools. The warranty applies for production and material break downs, and can only be used if the products have been in normal use. Please note that the normal

Seat with extra large hole: 310205/310205-B

Cleaning:

The product can be cleaned with regular cleaning without non-abrasive detergent. Wash and dry the product after the cleaning. The product can be washed up to 85° degrees in 3 minutes.

This product is CE marking which means the product complies with EU legislations and enables the free movement of the product in the European market.



See the spare parts on page 7

User guide for Shower/Commode chair M2 El-Tip with Joystick. Intented for use a bathing, showering and toilet visit.

(7/7)

Art. no	Spare Parts	Art. no	Spare Parts
310282	Back (Grey netweave)Chairs from Jan. 2011	800170	Clips, long
310252	Back (White plastic net)Chairs up to Jan. 2011	800169	Bolt for foot plate with nut
800326	Height, width adjustment bar	310226	Leg support (White nylon with velcro
800327	Screws for mounting of adjustment bar	800233	Upper frame
800546	Screws for mounting of adjustment bar for height, depth and width	800156	Bottom frame
800191	Grey Headrest cushion and mounting for the depth bar	800244	Ferule for round armrest, grey
800328	Depth adjustment bar	800012	Screws for fastening of armrest
800262	Vertical tube for headrest, height and depth adjustable (incl no. 7+8)	800082	Bolt for armrest
800027	Finger screw, grey	Electrical parts for the tip function	
800249	Grey ferule for headrest	800267	Remote control (including wire and plug)
310094	125 mm wheel with leg on. 1 pcs.	800269	Control box
800277	100 mm wheel with leg on. 1 pcs.	800270	Battery box incl. battery
310065	Clips, grey	800268	Actuator ("engine")
310165	Clips, magenta	800280	Transformer (Euro plug)
800013	Bush, wheel	800279	Transformer (British plug)
310340	Standardseat, grey	800281	Transformer (US/Japan plug)
310341	Seat lid for std. seat, grey	800272	Bolt with nut for fastening of actuator (on top)
310342	Seat with extra large hole, grey	800271	Bolt for fastening of actuator (below bottom)
310343	Seat lid for seat with extra large hole, grey	310264	Fastening screw, 4 pcs. (Fastening of the control box)
310352	Armrest for M2 with quick release & grey pu foam, right	Electrical parts for the motor function	
310352	Armrest for M2 with quick release & grey pu foam, left	800545	Joystick console
800181	Pu foam	800198	Control box
310076	White nylon bush for armrest	800195	Back wheel, set
310254	Quick release lock for armrests	800256	Protection box, right
800294	Bolt and mounting for fastening of frame	800287	Protection box, left
310276	Foot plate, right	800278	Universal transformer
310277	Foot plate, left	310264	Fastening screw, 4 pcs. (Fastening of the control box)

DJMed Pty Ltd WEB: djmed.com.au EMAIL: info@djmed.com.au